Achtung

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Caution

This product is restricted to use by or on the order of expert personnel / physicians.

Symbols

CE-sign and identification number of the Notified Body.

The product complies with the "Essential Requirements" of the Regulation (EU) 2017/745. Serial number

SN

Intended Use

The heating tube 7642HS in connection with the insufflator and the Option Therme serves for building up a pneumoperitoneum for diagnostic or operative Pelviscopy/Laparoscopy. The insufflation gas is preheated to body temperature by the heating tube. The heating tube has to be electrically operated with Insufflators with the Option Therme only. See also Instructions for use of the Insufflator.

Contraindications

Use of the heating tube is contraindicated, whenever Pelviscopy/Laparoscopy is contraindicated. The heating tube 7642HS may only be used in combination with a suitable Insufflator.

Important Hints

- Use heating tube 7642HS for its intended purpose only.
- The heating tube has to be used for heating gas (CO₂) only.
The heating tube is not suitable for warming up liquids.
- Prior to the operative Intervention, check all accessories as to their proper function. If faults are found or assumed, their use is strictly prohibited!
- Always use a sterile heating tube and a new filter for each patient.
- Interventions requiring the use of the products described in these instructions, may only be performed by persons with adequate training and experience.
- Prior to each Intervention, please consult the pertinent medical literature for techniques, complications and hazards.
- The user is fully responsible for observing the cleaning, disinfection and sterilization instructions. The manufacturer cannot be held liable for faults resulting from non-compliance with the above-mentioned regulations. Warranty conditions would not be applicable in such case.
- Independent from the general recommendations in this manual, the selection of accessories needed for the operative Intervention is within the sole responsibility of the operating surgeon.
- For avoiding cross-contamination, back-flow and contamination, the original filter has to be used. Filter and heating tube have to be replaced for each patient.
The filter is disposable and thus not re-sterilisable!
- Do not bend, squeeze or over-stretch heating tube and do not roll it up with a diameter smaller than 20 cm.

Service/Repairs/Modifications

In accordance with the international safety regulations for medical-technical devices, all activities such as check up's, inspections, repair, modifications etc. may only be carried out by the manufacturer or by explicitly authorized personnel.

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Cleaning, Disinfection and Sterilization

General

In order to maintain Performance of the heating tube and its accessories, they have to be cleaned, maintained and stored with adequate care. In order to avoid infections to the patient, the accessories coming into contact with human tissue have to be sterilized according to following procedure:

Step	Activity	
1	Cleaning	Automated Processing
2	Disinfection	
3	Sterilization	

Cleaning of heating tube

- Preliminary cleaning, immediately (within 2 hours after usage) when external contamination exists.
- Cleaning solution must be non-protein fixing.
- Rinse heating tube with running cold and warm water.
- Use a water pressure gun in order to rinse the lumen.
- Thorough cleaning with demineralized water.
- Drying of the parts with sterile cloths

Caution: Cleaning is to be carried out using a neutral or alkaline cleanser with a pH value of up to 11. Consult manufacturer specifications for concentration, contact time and temperature, adjusting these to suit the processing case in question. The cleaning solution must be suitable for silicone-rubber.

Disinfection of heating tube

Caution: Only parts carefully cleaned may be disinfected.

- Soak parts in solution and observe the manufacturer's instructions regarding solution concentration and soaking time. Extended soaking time or wrongful concentration may lead to damage at the parts.
- Do not pile up parts.
- Disinfectant residues should be rinsed off with sterile water under sterile conditions.
- Dry all parts with a sterile cloth, and wrap them in a sterile cloth afterwards.
- The parts must be stored in a closed, sterile Container.

Automated Processing

The heating tube can also be processed automatically.

- The processing should take place promptly after use, to prevent adhesions.
- The WD must be validated. By the validation of WD, it must be defined how many instruments and how they must be put into the WD.
- The below mentioned cleaning solution is to be understood as an example. Soak parts in solution and observe the manufacturer's instructions regarding solution concentration and soaking time.
- Suitability for silicone rubber must be provided.

We recommend the following schedule:

Nr.	Step	Water	Time [min]	Temp [C°]	Product	Dosage
1	Pre-clean	VE	2	20		
2	Clean	VE	10	55	neodisher® MediClean forte	10 ml/l
3	Rinse	VE	1	20		
4	Final rinse and thermal disinfection	VE	5	93		
5	Dry		15	70		

VE= fully desalinated water
WD= Washer- and Disinfector

CAUTION: The heating tube must not be soaked in disinfectant for longer than 30 minutes. Silicone absorbs disinfectants and would be damaged during subsequent steam Sterilization! Precede the subsequent steam sterilization, the heating tube must be dry inside

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WE RESERVED THE RIGHT FOR CHANGES ON TECHNICAL FEATURES AND DOCUMENTATION!

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and out.

Sterilization of heating tube

The parts to be sterilized should be cleaned, disinfected and dried. Packing into transparent sterile packs, transport after Sterilization and determined evaporation time are described in the Operator manual of the sterilizing unit.

We would recommend the following Sterilization methods:

Steam Sterilization in the steam sterilization device

- Steam sterilization device must be validated
- Die recommendations and the instructions of the device manufacturer should be followed
- HT has to be packed in materials suited for steam sterilization
- Double packaging in materials suited for steam sterilization is allowed
- Steam sterilization at triple fractionated prevacuum, at +134 °C, 3 bar abs. and hold time 5 minutes
- Drying at +120 °C for 20 minutes
- Sterilized pieces are to be placed in protection containers and stored in a suited area

We would recommend that the personnel responsible for sterilization, study the Operator manuals of the different sterilizing units. Please read the instructions for metal articles with lumen and porous articles with lumen. It is important to note that these recommended sterilization parameters are only valid, if the sterilizing equipment has properly been maintained and calibrated.


In accordance with the above-mentioned conditions you can resterilize the heating tube a maximum number of 200 times.

Description of heating tube connection

see also the drawing shown below and the Instructions for use of the devices System Insufflator and Flow Therme

1. Connection socket for the gas
2. Connection plug of the heating hose cable
3. Gas-output fitting to the patient at the insufflator
4. Filter (disposable)
5. Filter adaptor
6. Gas-Coupling fitting at the heating hose
7. Gas heating hose
8. Heating hose coupling to instrument
9. Veress needle

Electrical Connection of the Heating Tube: 19,5 V / 1 A DC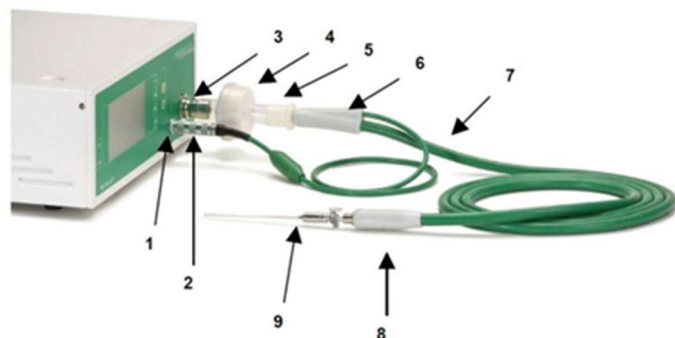

Disposal of the heating tube

The heating tube must be put through the cleaning process before being disposed. After the cleaning process it must be disposed through the clinical disposal system.

Warranty

See out General Terms and Conditions.

In case of unauthorized opening / modifications and/or repairs of the heating tube we cannot be held liable for proper and safe function of the device.

All warranty claims are declared null and void then.